

Taking OFF & Reuse

Face Shield Usage

Cleaning & Disinfecting




① **Apply Disinfectant***
wipe front and back of
shield with paper towel




② **Wipe** elastic strap




③ **Wipe** foam head band

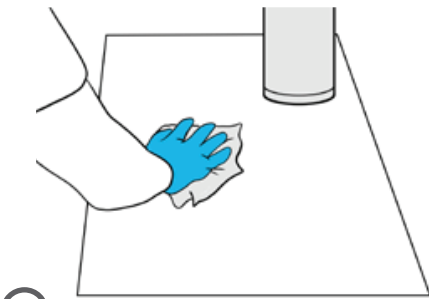


④ **Wipe** area on table
with disinfectant

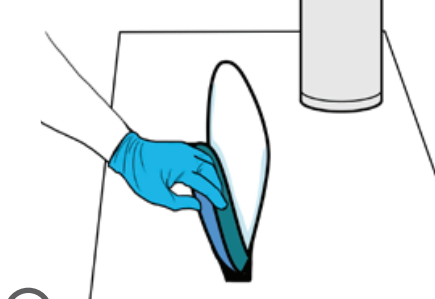

⑤ **Place** shield upside
down to dry or in an
open bag

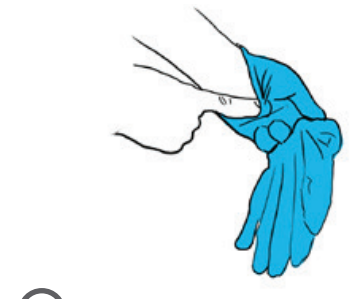

⑥ **Remove** gloves

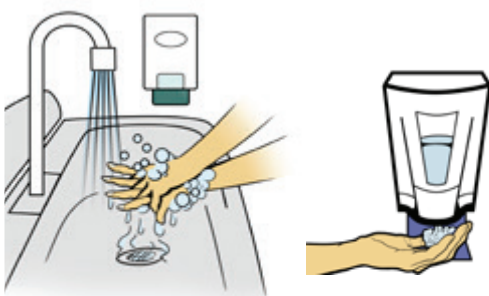

⑦ **Wash hands** with soap and
water for 20 seconds or
Sanitize hands

***Disinfectant** supplied by your facility
location or MSU Clean Cat Kits*

***Glove** use not required unless it's a facility
specific standard operating procedure*